ULTIMED SHARPS DISPOSAL SYSTEM

PACKAGE COMPONENTS

- Prepaid return shipping box
- Brown inner box
- 100 insulin syringes
- UltiGuard® Safe Pack sharps container
- Tracking manifest
- Instructions for use
- Reclosable plastic bag
- Tape strip

DO NOT THROW AWAY ANY SHIPPING OR PACKAGING MATERIALS

GETTING STARTED

1. Remove the white prepaid return shipping box from the box you received it in. **DO NOT** discard the white box or its contents. This will be needed to return your system for disposal.
2. Carefully open the white prepaid return shipping box and the brown inner box. **DO NOT REMOVE** the brown inner box from the white box.
3. Remove the UltiGuard Safe Pack sharps container.
4. Keep all packaging materials and this instruction sheet in a clean, dry place for later use.

INSTRUCTIONS FOR USE

For up to date instructions, visit www.disposemyneedles.com.

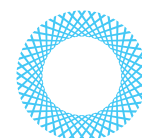
1. Follow the UltiGuard Safe Pack instructions for use on the back of the container.
2. When you have used all the insulin syringe supply, place the Safe Pack into the reclosable plastic bag and return to the brown inner box by following the instructions on the reverse side of this page.
3. Do not place medications in this container.
4. Maximum weight: 5 lbs.

WARNINGS

- Failure to properly complete and sign the tracking manifest may result in enforcement under state, local, and federal rules and regulations and additional fees may be applied.
- Avoid exposing container and shipping materials to extreme heat.
- Keep in upright position and out of reach of children.
- Do not overfill. Disposable single use sharps container for home use only.
- Do not use for disposal of any sharps not packaged with this container.

For assistance, you may contact us at our website at www.disposemyneedles.com or call 1-844-8SHARPS (1-844-874-2777).

UltiGuard® is a registered trademark of UltiMed, Inc. © 2021 UltiMed, Inc.

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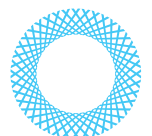
DO NOT THROW AWAY ANY SHIPPING OR PACKAGING MATERIALS

ULTIMED SHARPS DISPOSAL SYSTEM

INSTRUCTIONS: Preparation for Shipping and Disposal

- 1.** Open packaging materials and remove UltiGuard Safe Pack.

- 2.** Save and store packaging materials for return shipping.

- 3.** When you have used all the insulin syringe supply, and sharps are safely contained, place the Safe Pack into the reclosable plastic bag and seal.

- 4.** Put sealed plastic bag with Safe Pack into the brown inner box.

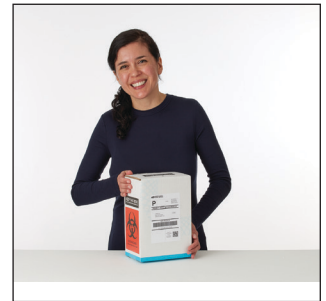
- 5.** Close the brown inner box and prepaid return shipping box following the numbered flaps. Seal the closure tab of the prepaid return shipping box with provided tape strip.

- 6.** Carefully remove tracking manifest from the plastic bag on the side of the prepaid return shipping box and complete.


- 7.** Fill in tracking manifest completely, retaining the dark yellow copy for your records. Place tracking manifest back into the bag on the side of the prepaid return shipping box.

- 8.** Mail the properly sealed kit through the U.S. Postal Service. Take it to the nearest U.S. Postal Office or place in your mailbox for pick up. Postage is prepaid.

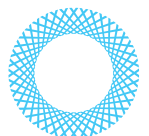For assistance, you may contact us at our website at www.disposemyneedles.com or call 1-844-8SHARPS (1-844-874-2777).

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